


Extraction set for hip revisions

Jig-guided system


fitting **Corail®** (DePuy Synthes)

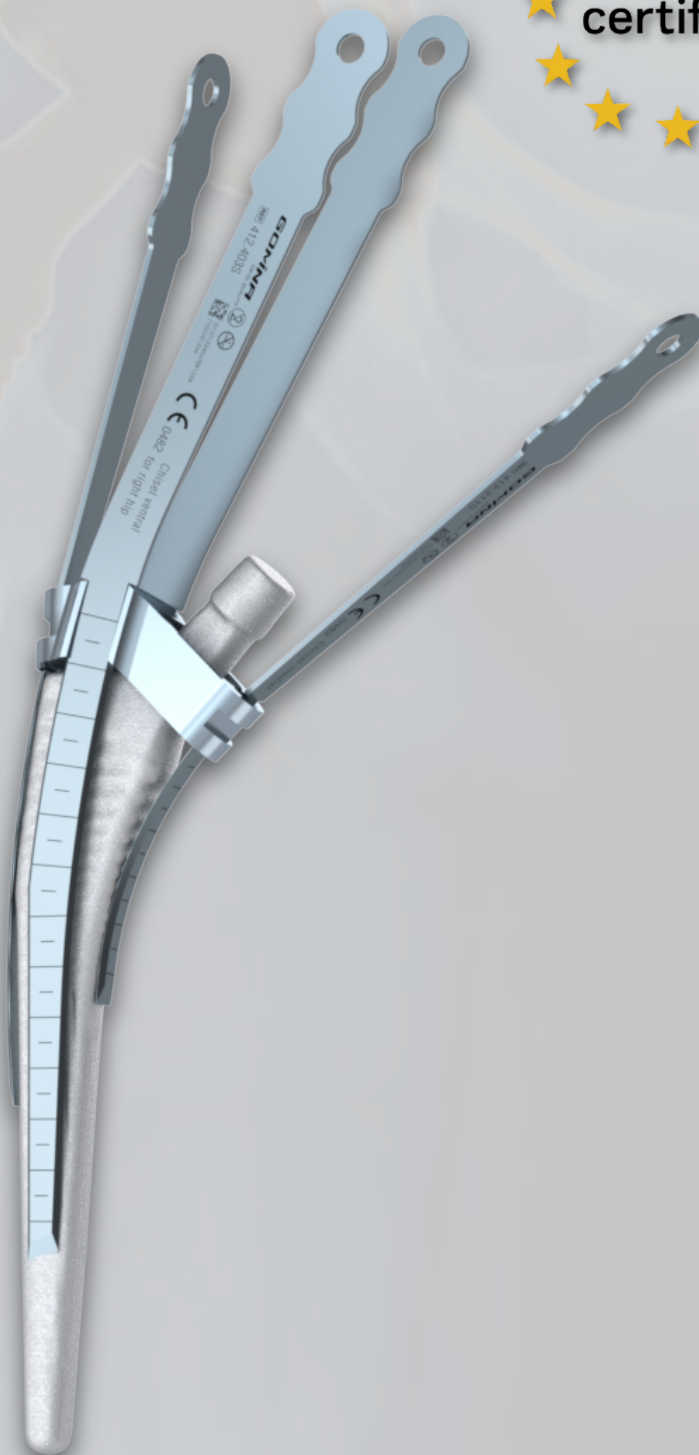

Official Distributor
for UK

Handle and extraction bolt


The extraction bolt can be attached on the top or bottom of the handle.

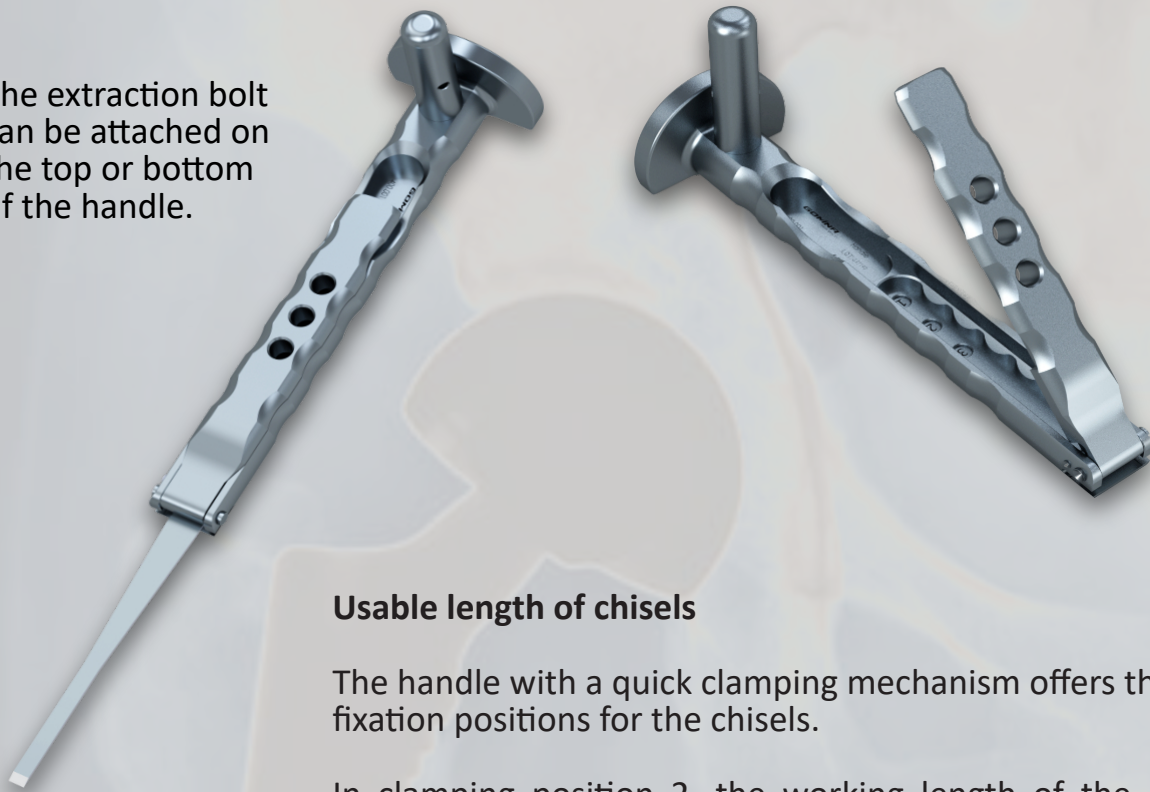

Usable length of chisels

The handle with a quick clamping mechanism offers three fixation positions for the chisels.

In clamping position 2, the working length of the chisel is extended by 15 mm compared to position 1, and by a further 15 mm in position 3. In total, the working length can be extended by 30 mm from position 1 to position 3.


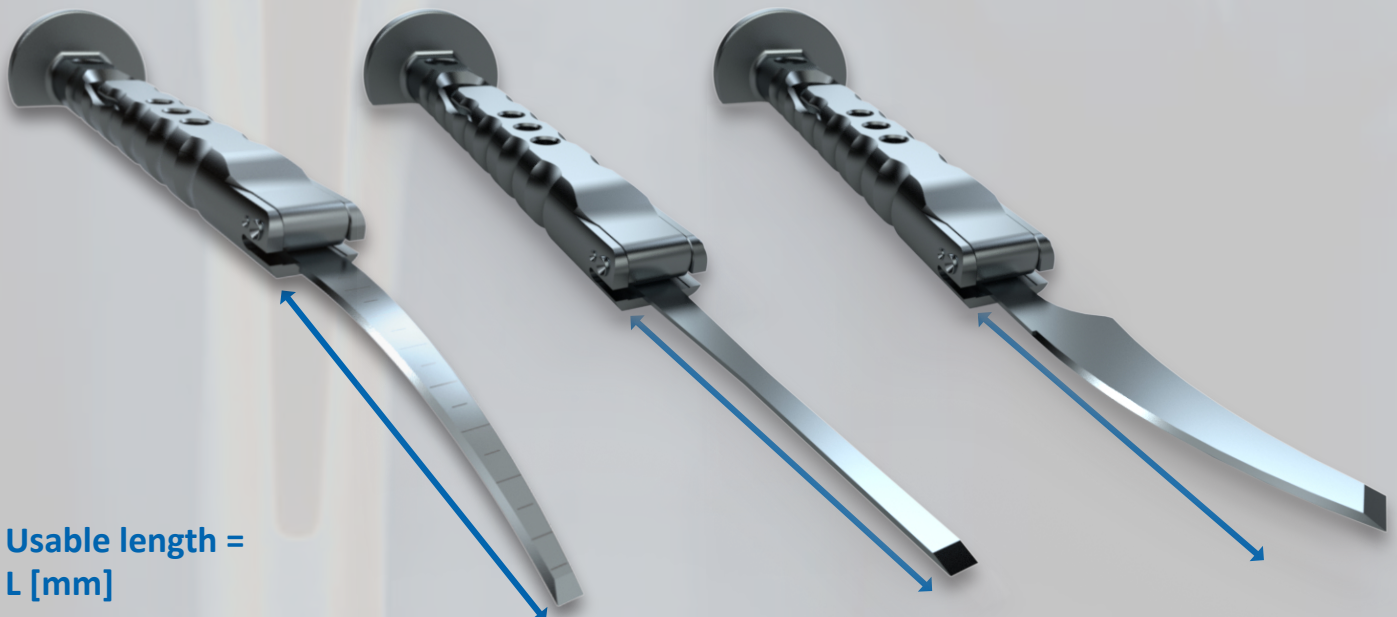


Usable length =
L [mm]

Jig

Guide slots

Guide grooves

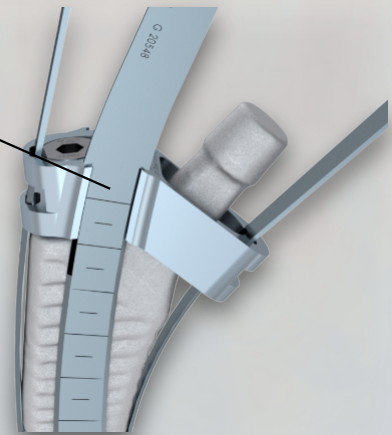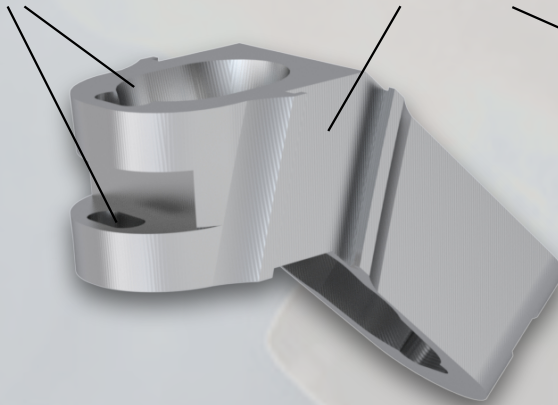

The jig-guided chisels are matched to the guide slots and grooves of the corresponding jigs. The chisels are guided through both guide slots (or grooves) until the cutting edge makes contact with the bone. Then the chisels can be driven in with light hammer strikes.


The jig is positioned on the cone of the stem and attached with the fixation screw.



The jig must be correctly positioned on the stem in the lateral-medial direction before it is fixed with the screw. Use the lateral guide slot as a reference.

Overview of chisels with jig

**Note: colors are only for illustrative purpose. The products are not color-coded.*

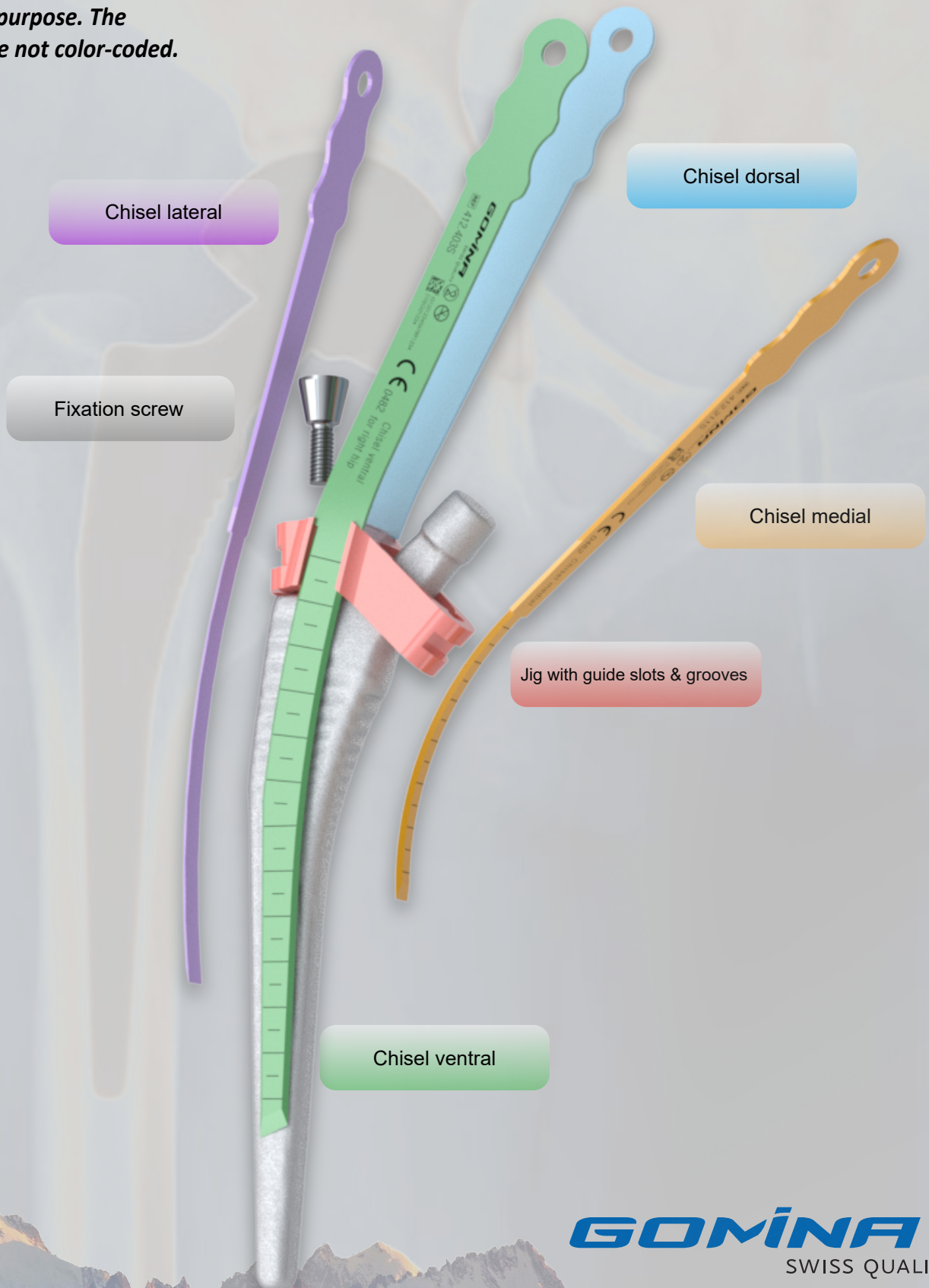

GOMINA
SWISS QUALITY


Indicated references by stem size

Corail® stem sizes **08, 09, 10**

(for Corail 135° STD, 125° STD, 135°SN, all collarless)



Reference (sterile)	Article Description	L [mm]	W [mm]	T [mm]
412.201S	Chisel lateral	133.0	4.8	1.19
412.211S	Chisel medial	105.2	5.8	1.19
412.401S	Chisel ventral for right hip	170.0	4.0	1.19
412.402S	Chisel dorsal for right hip	170.0	4.0	1.19

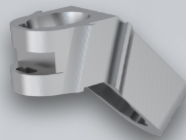

Ref. (non-sterile)	Article Description
412.901	Jig
400.102	Fixation screw
400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key




GOMINA
SWISS QUALITY


Indicated references by stem size

Corail® stem sizes **11, 12, 13**

(for Corail 135° STD, 125° STD, 135°SN, all collarless)

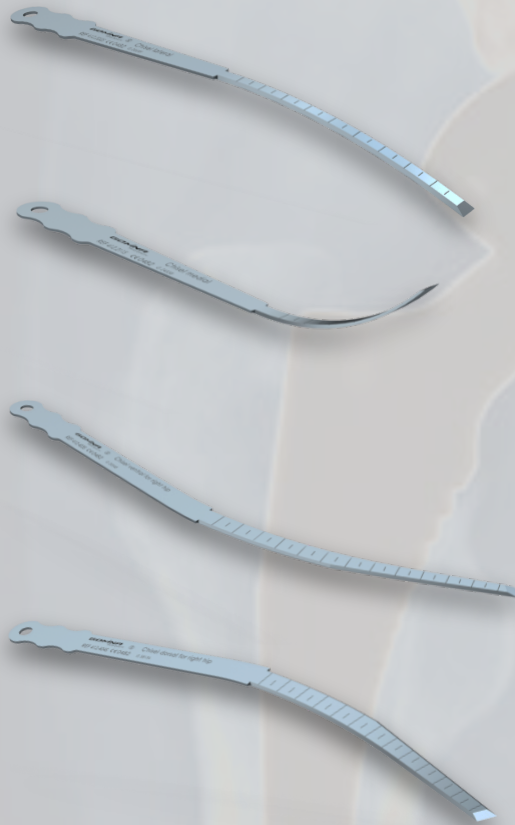

Reference (sterile)	Article Description	L [mm]	W [mm]	T [mm]
412.202S	Chisel lateral	138.0	6.0	1.19
412.211S	Chisel medial	105.2	5.8	1.19
412.403S	Chisel ventral for right hip	170.0	7.1	1.19
412.404S	Chisel dorsal for right hip	170.0	7.1	1.19

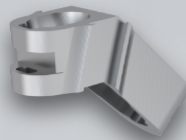

Ref. (non-sterile)	Article Description
412.903	Jig
400.102	Fixation screw
400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key



GOMINA
SWISS QUALITY


Indicated references by stem size

Corail® stem sizes **14, 15, 16**

(for Corail 135° STD, 125° STD, 135°SN, all collarless)



Reference (sterile)	Article Description	L [mm]	W [mm]	T [mm]
412.202S	Chisel lateral	138.0	6.0	1.19
412.211S	Chisel medial	105.2	5.8	1.19
412.405S	Chisel ventral for right hip	215.3	7.4	1.19
412.406S	Chisel dorsal for right hip	215.3	7.4	1.19

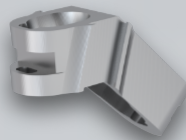

Ref. (non-sterile)	Article Description
412.905	Jig
400.102	Fixation screw
400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key




GOMINA
SWISS QUALITY

Indicated references by stem size

Corail® stem sizes 18, 20


(for Corail 135° STD, 125° STD, 135°SN, all collarless)

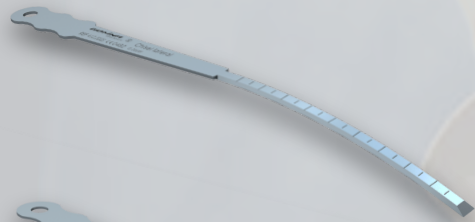

412.202S

Chisel lateral


L [mm] W [mm] T [mm]
138.0 6.0 1.19

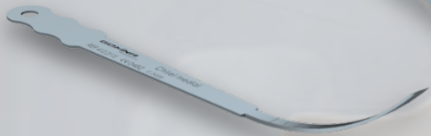

412.211S

Chisel medial


105.2 5.8 1.19

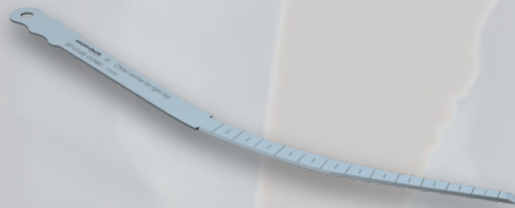

412.407S

Chisel ventral
for right hip


233.5 9.7 1.19

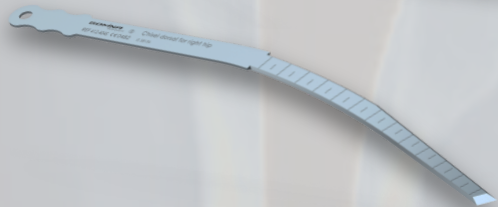

412.408S

Chisel dorsal
for right hip

233.5 9.7 1.19

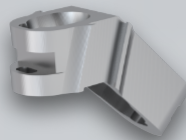

412.907

Jig



400.102

Fixation screw



400.104 (short)

400.107 (long)

400.112 (with handle)

Allen key




GOMINA

SWISS QUALITY


Indicated references by stem size

Corail® stem sizes **08, 09, 10**

(for Corail High Offset 135° KHO, 125° KLA, all collarless)

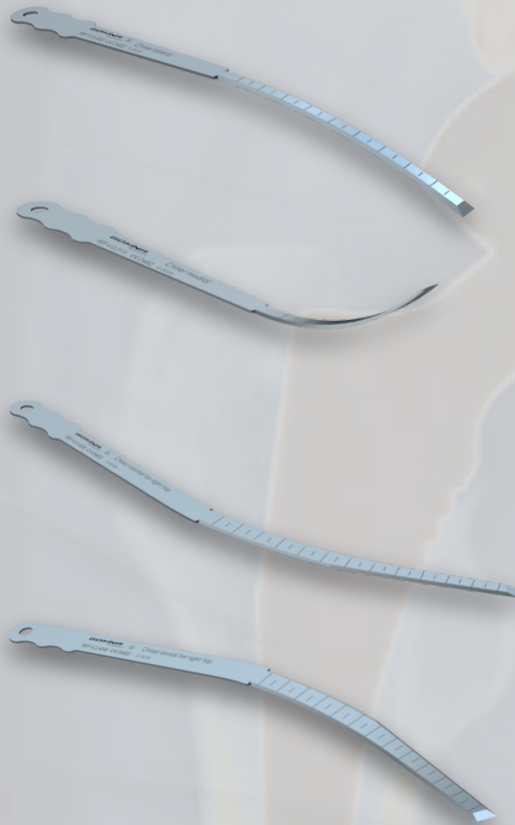

Reference (sterile)	Article Description	L [mm]	W [mm]	T [mm]
412.201S	Chisel lateral	133.0	4.8	1.19
412.211S	Chisel medial	105.2	5.8	1.19
412.401S	Chisel ventral for right hip	170.0	4.0	1.19
412.402S	Chisel dorsal for right hip	170.0	4.0	1.19

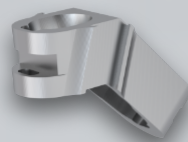

Ref. (non-sterile)	Article Description
412.909	Jig
400.102	Fixation screw
400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key




GOMINA
SWISS QUALITY


Indicated references by stem size

Corail® stem sizes **11, 12, 13**

(for Corail High Offset 135° KHO, 125° KLA, all collarless)

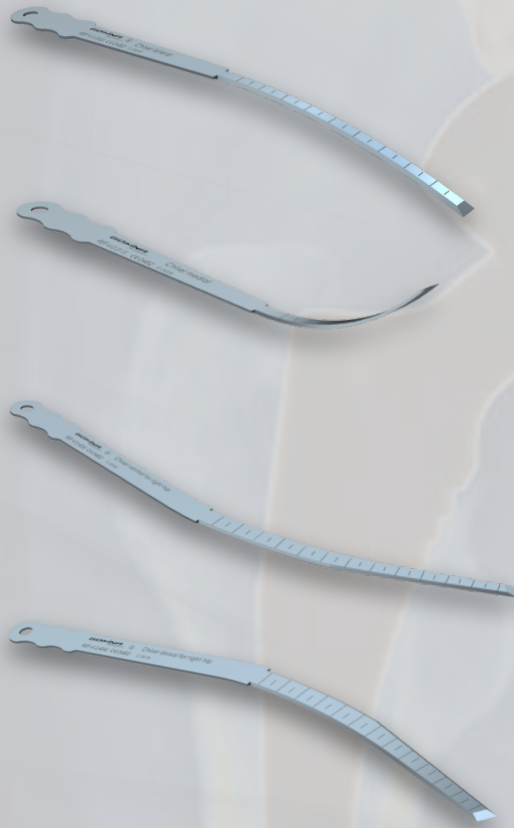

Reference (sterile)	Article Description	L [mm]	W [mm]	T [mm]
412.202S	Chisel lateral	138.0	6.0	1.19
412.211S	Chisel medial	105.2	5.8	1.19
412.403S	Chisel ventral for right hip	170.0	7.1	1.19
412.404S	Chisel dorsal for right hip	170.0	7.1	1.19

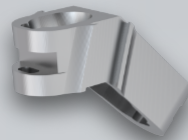

Ref. (non-sterile)	Article Description
412.910	Jig
400.102	Fixation screw
400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key




GOMINA
SWISS QUALITY


Indicated references by stem size

Corail® stem sizes **14, 15, 16**

(for Corail High Offset 135° KHO, 125° KLA, all collarless)

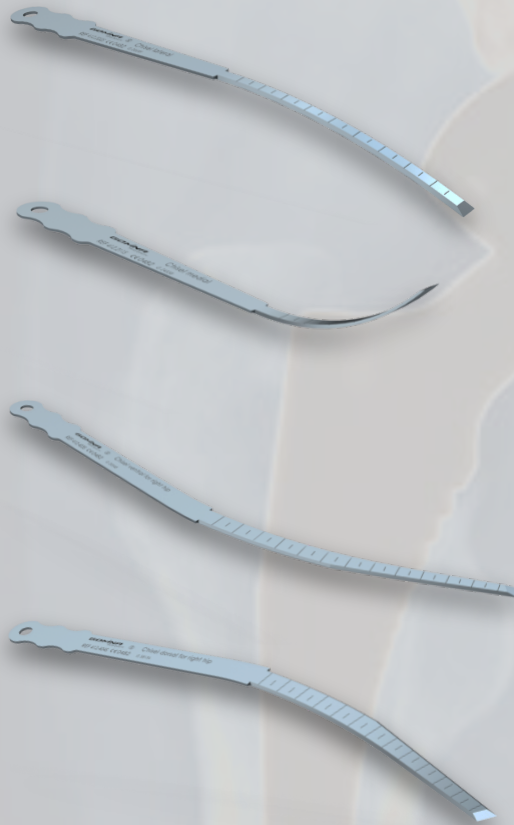

Reference (sterile)	Article Description	L [mm]	W [mm]	T [mm]
412.202S	Chisel lateral	138.0	6.0	1.19
412.211S	Chisel medial	105.2	5.8	1.19
412.405S	Chisel ventral for right hip	215.3	7.4	1.19
412.406S	Chisel dorsal for right hip	215.3	7.4	1.19

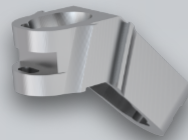

Ref. (non-sterile)	Article Description
412.911	Jig
400.102	Fixation screw
400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key




GOMINA
SWISS QUALITY


Indicated references by stem size

Corail® stem sizes **18, 20**

(for Corail High Offset 135° KHO, 125° KLA, all collarless)

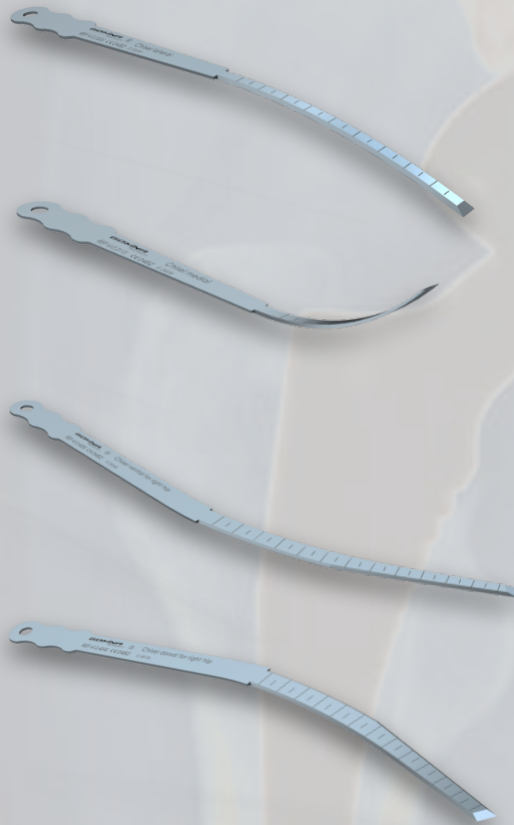

Reference (sterile)	Article Description	L [mm]	W [mm]	T [mm]
412.202S	Chisel lateral	138.0	6.0	1.19
412.211S	Chisel medial	105.2	5.8	1.19
412.407S	Chisel ventral for right hip	233.5	9.7	1.19
412.408S	Chisel dorsal for right hip	233.5	9.7	1.19


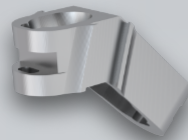


Ref. (non-sterile)	Article Description
412.912	Jig
400.102	Fixation screw
400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key



GOMINA
SWISS QUALITY


Overview of all references

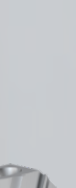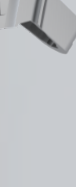	Corail® stem sizes	Reference	Article Description	L [mm]	W [mm]	T [mm]
	All sizes	400.001	Handle incl. extraction bolt			
	All sizes	400.100	Extraction bolt (spare part)			
	08/09/10	412.201S	Chisel lateral	133.0	4.8	1.19
	11/12/13/14/15/16/18/20	412.202S	Chisel lateral	138.0	6.0	1.19
	All sizes	412.211S	Chisel medial	105.2	5.8	1.19
	08/09/10	412.401S	Chisel ventral for right hip	170.0	4.0	1.19
	08/09/10	412.402S	Chisel dorsal for right hip	170.0	4.0	1.19
	11/12/13	412.403S	Chisel ventral for right hip	170.0	7.1	1.19
	11/12/13	412.404S	Chisel dorsal for right hip	170.0	7.1	1.19
	14/15/16	412.405S	Chisel ventral for right hip	215.3	7.4	1.19
	14/15/16	412.406S	Chisel dorsal for right hip	215.3	7.4	1.19
	18/20	412.407S	Chisel ventral for right hip	233.5	9.7	1.19
	18/20	412.408S	Chisel dorsal for right hip	233.5	9.7	1.19
	08/09/10	412.901	Jig (for Corail 135° STD, 125° STD, 135°SN, all collarless)			
	11/12/13	412.903	Jig (for Corail 135° STD, 125° STD, 135°SN, all collarless)			
	14/15/16	412.905	Jig (for Corail 135° STD, 125° STD, 135°SN, all collarless)			
	18/20	412.907	Jig (for Corail 135° STD, 125° STD, 135°SN, all collarless)			
	08/09/10	412.909	Jig (for Corail High Offset 135° KHO, 125° KLA, all collarless)			
	11/12/13	412.910	Jig (for Corail High Offset 135° KHO, 125° KLA, all collarless)			
	14/15/16	412.911	Jig (for Corail High Offset 135° KHO, 125° KLA, all collarless)			
	18/20	412.912	Jig (for Corail High Offset 135° KHO, 125° KLA, all collarless)			
	All sizes	400.102	Fixation screw			
	All sizes	400.104 (short) 400.107 (long) 400.112 (with handle)	Allen key			

Contact



Gomina AG
Raiftstrasse 4
3989 Niederwald, Switzerland
Phone +41 27 970 12 70
Fax +41 27 970 12 75
www.gomina.ch
sales@gomina.ch

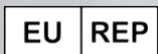

GOMINA Deutschland GmbH
Ludwig-Vierling-Strasse 7
89233 Neu-Ulm, Germany
info@gomina.de

Version 2026-03
Last modification: 27.03.2026

Official Distributor for UK:

Judd Medical Limited

Unity House
Buntsford Park Road
Bromsgrove
Worcestershire
B60 3DX
Tel: 01527 559010
sales@judd-medical.co.uk
www.judd-medical.co.uk

